# AP721791-10

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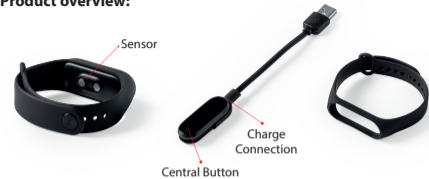
# AP721791-10 Droy thermometer Smart watch User manual

It is recommended to read this user manual carefully and keep it for future reference.

## Product overview:

## Specifications:

- Bluetooth Version: 4.0
- Screen: 0.96"
- Charging Voltage: 4.2V DC
- Li-polymer battery 90mAh 3.7V
- Operating Distance: Up to 10 meters
- Working Time: up to 3 days
- Compatible with iOS 9.0 and Android 5.0 or newer
- Degree of Protection: IP65
- Frequency range: 2.4GHz
- Max RF power: 4dbi
- Input: 4.2V, 90mAh
- Output : 3.0V, 0.9mAh
- Temperature: -10°C~45°C

## Operation:

### Synchronize the watch with your mobile device through the application:

- Download "Lefun Health" from APP Store (iOS) or Google Playstore (Android). Also available from the QR code shown below.

- Hold down the main button to power on the watch.
- Turn on the Bluetooth and location on your device.
- Open the APP on your device (keep the watch near your phone while pairing).
- Pair the watch with your device by following these steps

- After synchronizing you can personalize your APP by entering your personal details and you can set your daily workout goals.
- Explore the APP to learn about the different functions.  
Note: After synchronizing, you can disconnect the location.

**Note:** To use some of the APP functions, firstly you will need to register on the application.  
**Note:** For the APP to work correctly you will need to agree to the APP permissions.  
**Note:** Any updates may affect the APP functions.

## Functions:

### Sleep Mode

This function can automatically detect and recognize your sleeping status and yours monitors whole sleep progress based on the movements you make during the night.  
The measurement time is from 22:00h and you can see the result in the APP.

### Measurements

This device allows you to measure the heartbeat, the temperature, the blood pressure and blood oxygen percentage.

Press the main button until you get to the desired measurement screen, then hold down the button.  
From the back of the bracelet, a green light beam will appear.  
Now you can start measuring.

Blood pressure, heartbeat and blood oxygen

Temperature

**Note:** You can start the measurement from the APP.

The temperature reading is taken automatically every 5 minutes.  
You can also take the temperature manually, selecting either body temperature or surface temperature. To change the temperature measurement mode press the centre button for a few seconds

The results displayed on the watch are synchronized automatically in the APP.  
Note: Automatic synchronization may take several minutes.

### Notifications and Alarms

This device allows you to set different alarm types, such as message notify, call reminder and Application notify.  
You can also set the alarm clock or sedentary reminder.

This device stores the last notification.  
To see it, press the central button of the watch until you reach the screen then keep it pressed down.

Note: Once the notification has been seen, the message will disappear.

### Exercise Mode:

This watch calculates steps, walking distance and calorie consumption.  
Press the main button until the following screen appears, then keep it pressed down to access. Press the button to see the different options.

**Note:** Watch data will start from zero every day at 00:00h

In addition, the device has the function "sports", which allows you to monitor sports activities such as running, jump rope or sit-ups. To do this, press the central button until you reach to corresponding screen and keep it pressed to access. Press the button to see the different options and keep it pressed again to access. At this moment, the timer will start. To finish, press and hold down the central button.

### Note:

To change the layout of the main clock screen, press the central button until you see this icon.

Keep the button pressed down to access then keep pressing until you reach.

Press and hold to access and keep pressing to see the three available options.  
To enter an option, press and hold down the central button on the desired option.

### Power off the watch:

In the 'More' option, press the central button until you reach the power icon and keep it pressed.

### Reset the watch:

In the 'More' option, press the central button until you reach the reset icon and keep it pressed.

## Charging:

Connect the USB connection of the device to an appropriate power outlet.

## Warnings:

- Do not expose this product to rain or damp conditions.
- Prevent this item from falling as it could cause serious damage.
- Do not dismantle, repair or modify this product by yourself, contact an authorized specialist.
- In order to avoid any risk, do not dispose of this product inappropriately or throw it into the fire as there is a built-in Lithium battery.
- Clean it with a dry cloth.
- Do not place this item near sources of heat or expose to direct sunlight.
- Do not insert any metal objects into this device as there is a risk of causing a short circuit in the unit.

This item is in conformity with the following standards:  
RoHS Directive 2011/65/EU and its subsequent amendments Directive (EU) 2015/863  
Radio Equipment Directive (RED) - Directive 2014/53/EU  
UK Electrical Equipment (Safety) Regulations (SI 2016/1101)  
UK Radio Equipment Regulations (SI 2017/1206)

CAUTION: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

This symbol both in the product and its user manual indicates that at the end of the devices electrical life, it should be recycled separately from your household waste. There are adequate means to dispose of this material for proper recycling. For more information, contact your local authorities.

Distributor: Cool Collection, 1476 Budapest, POB:176. Hungary  
Registration nr.: 01-09-276783  
Made in China